

EMT

EMERGENCY &
MILITARY TOURNIQUET

HOSPITAL PERFORMANCE FOR MEDICS ON THE MOVE

Delfi's unrivalled **Emergency & Military Tourniquet** is designed to deliver the highest level of tourniquet safety with durable portability. In ground-based tactical situations and first response trauma incidents, the EMT is used as a life-saving haemorrhage control device.

Based on over 40 years of Delfi's world leading tourniquet technology, the EMT's wide bladder pneumatic design is proven to completely stop arterial blood-flow on both upper and lower limbs, unlike narrow tourniquets which are often ineffective no matter how tightly they are applied, potentially causing further injury to the limb.

An independent assessment completed by the United States Army Institute of Surgical Research (USAISR) found the EMT to be effective in 100% of all trials. In addition, the EMT was found to be "wider and thus significantly less painful than any device tested and is much less likely to induce nerve damage".

EMERGENCY

POLICE

FIRE / RESCUE

INTUITIVE RELIABLE DESIGN

Durable aluminum clamp and superior materials designed to withstand the elements. No hook or loop is used which can become ineffective if contaminated with mud and dirt especially in field applications

PERSONALIZED FIT

Provides close conformance to a wide range of upper and lower limb shapes and sizes, with a circumference range from 3 in. (8cm) to 34 in. (86cm)

OPTIMIZED WIDTH

A wider 3.5 in. (8.9 cm) bladder width results in the need for lower pressure to fully occlude the limb, making it far less painful and less likely to cause nerve damage

SPEED AND EASE

Fast and easy one hand application for self-aid situations and crucial time savings. Easy slip-on design can also be wrapped around if distal end of limb is inaccessible. Requires less than 30 seconds to apply and inflate!

LIGHTWEIGHT AND COMPACT

Space-saving lightweight design ideal for any emergency kit and medical workers on the move. Weight 220 grams (7.8 oz), rolled up size 5 x 2.75 x 2.5 in. (13 x 7 x 6.4 cm)

SAFE AND RAPID INFLATION

A pneumatic pump delivers fast and safe tourniquet pressure, inflating only the used length of the bladder until blood flow is stopped



Part No. 9-1100-001

 **Delfi**

EMT

EMERGENCY &
MILITARY TOURNIQUET

PROVEN 100% EFFECTIVE
AND RELIABLE FOR
SAFETY IN THE FIELD

THE ONLY LIFE-SAVING
TOURNIQUET YOU WILL
EVER NEED TO SAFELY STOP
HAEMORRHAGING

- Fast, easy one-handed application
- Safely controls pressure via pneumatic pump
- Optimal cuff width for safety and close conformance
- Light weight and compact
- Robust design and components

MEDICAL
EVACUATION VEHICLES

COMBAT
MEDICS

ECHELON I-III
MEDICAL FACILITIES



“The EMT, because it is pneumatic, can be tightened (inflated) in very small increments. Thus it is safer due both to its greater width, as well as the ability to more finely adjust pressure. The pressure profile between a tourniquet and underlying tissue is less damaging”.

As recommended by the United States Army
Institute of Surgical Research (USAISR)

About Us

Delfi Medical Innovations is a world leader in advancing tourniquet safety and technology. Members of Delfi's team have been exclusively involved in tourniquet development and evaluation for more than 40 years.

Delfi's patented tourniquet instruments and tourniquet cuffs have been designed to help surgeons and operating-room nurses provide personalized safe and effective care for their patients.

Contact Us

For sales enquiries and ordering information contact Customer Service at:

1.800.933.3022 (US & Canada)
+ 604.742.0600 (Elsewhere)
sales@delfimedical.com

Delfi Medical Innovations Inc.
106 - 1099 West 8th Avenue
Vancouver, BC Canada V6H 1C3

To learn more on how our tourniquet systems help provide
personalized safe and effective care for your patients visit:

delfimedical.com